


Implant Screw Driver SD-T Driver

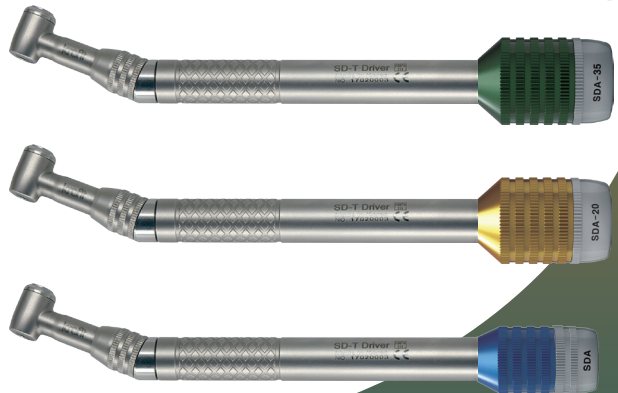


- Please be sure to read this manual before operation.

1. Instruction for Use

1) Preparation before use

- This product should be used after sterilization.
- The procedure should be performed only by a specialist such as a dentist.
- Sterilization method
 - ① First, remove the contra-angle head (a) & handle (b) and clean.
 - ② Sterilize the contra-angle head part (a) under 132°C for 4 minutes.

2) Product composition

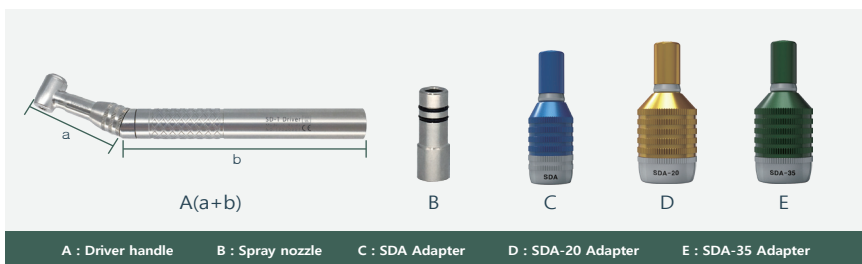

3) How to use

- Supposed to be use proper driver tip as same shape screw of implant abutment
- Rotate 'handle knob' of the adapter by fingers to the direction of right and left to tighten or loosen screw.

4) How to clean

- Be sure to use oil spray after cleaning.
- First, connect the spray nozzle (B) to the driver handle (A).
- Spray lubricant oil for 2-3 seconds as shown.
- After cleaning, wipe lubricant oil spilled over the surface of the product and keep it at room temperature for drying.



2. Cautions

1) Precautions for use

- At each use, sterilization and oil spray cleaning recommended according to the prescribed method.
- The temperature of sterilization should not exceed 132 °C maximum.

2) How to use adapter

- Insert the screwdriver until it clicks to the end of the screwdriver.
- As to SDA-20 adapter, when the torque reaches 20 Ncm, it becomes idle.
- As to SDA-35 adapter, when the torque reaches 35 Ncm, it becomes idle.
- As to SDA adapters, the torque can not be adjusted, so if excessive force is applied, it may cause damage to inside part of the product.



saeyang

Head Office | (Galsan-dong), 348, Seongseo-ro, Dalseo-gu, Daegu, 42697, Korea

www.saeyang.com
marathon@saeyang.com

TEL +82-53-582-9001 FAX +82-53-581-9003

2017-SDTDRIIVER-EN-01

Oct, 2018
Rev. 2-E